paraTest®

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Replace the blue insert in the container, close cap and take to your veterinary surgeon or laboratory as appropriate.

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In case of overflow, the flotation solution will be collected in the outer ring of the insert.

easy handling, clean, precise

Ritter parasite diagnostic system guarantees the pet owner easy and clean handling when taking samples of fecals and the veterinarian/laboratory staff problem-free, hygienic analysis of fecals with reliable, accurate results.

HOW PARATEST WORKS

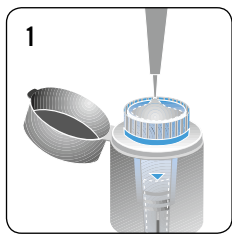
Suspending the fecal sample in the flotation medium which has a specific weight of 1.2 the ova float to the top of the insert. The heavier parts of the sample remain on the bottom of the container. The ova then can be taken/transferred to a slide for micro-scopic examination. Flotation medium is sodium nitrate solution (31%).

ADVANTAGES

- High stability
- Overflow provision
- Simple opening/closing of container
- Consumer friendly design

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
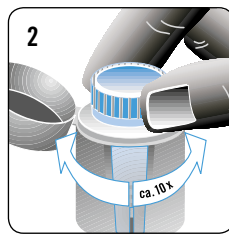
Lift the cap and fill the blue insert with flotation solution to the middle mark.

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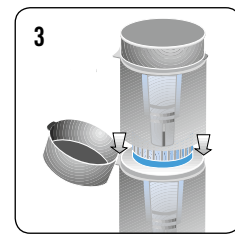
Rotate the blue insert back and forth approx. ten times to separate the parasite ova from the fecal sample.

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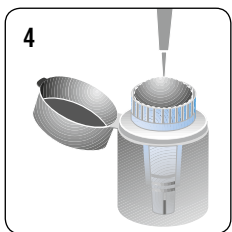
To secure the blue insert in the outer container, press down into place by applying pressure to a second container placed on top.

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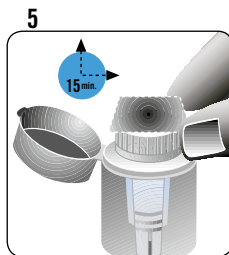
Completely fill the blue insert with additional flotation solution to form a meniscus.

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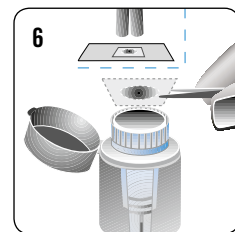Float a 22 mm slip on the meniscus for 15 mins. By means of flotation the parasite ova if present will remain on the cover slip.

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Remove cover slip carefully and start the microscopic examination at 100 x magnification maximum.

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Parasite diagnostic system – easy handling, clean, precise

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